Produktbeschreibung

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SMOOTHSTAR PUTTY SOFT YELLOW

Putty vinyl silicone impression material

Instructions for Use

MD

Product description

SMOOTHSTAR PUTTY SOFT YELLOW is a kneadable base impression material consisting of addition curing vinyl silicones. In conjunction with A-MEDIUM SUPERHYDRO (Fast Set) or Smoothstar Mono Fast lila it can serve as an ideal base for the double-mix technique (1 step/2 phase).

SMOOTHSTAR PUTTY SOFT YELLOW is a soft but not sticky, kneadable precision impression material. It is easy to mix and shows good setting properties.

SMOOTHSTAR PUTTY SOFT YELLOW meets the requirements of ISO 4823, **type 0** (putty consistency).

Indications/Intended use

- Double-mix technique (1 step/2 phase) with e.g. SMOOTHSTAR LIGHT NORMAL PINK
- Correction impression technique (2 step/2 phase) with e.g. SMOOTHSTAR LIGHT NORMAL PINK or SMOOTHSTAR LIGHT FAST LILA

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

Perforated or non-perforated metal or plastic trays may be used with SMOOTHSTAR PUTTY SOFT YELLOW. In order to improve adhesion of SMOOTHSTAR PUTTY SOFT YELLOW to the tray, a suitable silicone adhesive should be applied according to the corresponding instructions before loading the impression tray.

Use equal parts (volume or weight) of base paste and catalyst paste. Take the base paste with the dark measuring spoon and the catalyst paste with the white measuring spoon. Knead until the colour is homogeneous (at least **30 seconds**).

Impression taking

Double-mix technique

Load the impression tray with SMOOTHSTAR PUTTY SOFT YELLOW and apply SMOOTHSTAR LIGHT NORMAL PINK around the preparation. Tray loading and syringing around the preparation have to be completed at the same time. Insert the tray not later than **1:30 minutes** from start of mixing (working time at 23°C or 74°F) into the mouth. Maintain position for at least **3:00 minutes**.

Two-step (correction) impression technique

Load the impression tray with SMOOTHSTAR PUTTY SOFT YELLOW. Insert the tray not later than **1:30 minutes** from start of mixing (working time at 23°C or 74°F) into the mouth. Maintain position for at least **3:00 minutes**.

Cut drain channels into the SMOOTHSTAR PUTTY SOFT YELLOW impression. Load the auto mixed SMOOTHSTAR LIGHT NORMAL PINK or SMOOTHSTAR LIGHT FAST LILA onto the tray with the preliminary impression and/or directly onto the teeth. Position the tray within the working time of the correction impression material into the mouth. Maintain position until hardening has finished.

Refer to the corresponding user instructions.

The impression should be stored at normal room temperature (max. 25 °C or 77 °F).

Casting

The impression can be poured **30 minutes** after removal from the mouth. No other time limitation does exist.

Electroplating

All impressions can be copper- or silverplated.

Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones.

Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not interchange lits of pots. Do not interchange measuring spoons.
- Do not use with standard (condensation reaction type) silicone impression materials.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- When using the double-mix technique (1 step/2 phase), make sure that both materials have the same setting time in mouth. The working time is determined by the material with the shorter working time.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

P. L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P. L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P. L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P. L. Superior Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, P. L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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P. L. Superior Dental Materials GmbH, Stiller Weg 15 A 22607 Hamburg, Germany