# CAVEX IMPRESAFE

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**EN (English)**

**INTENDED PURPOSE:** Cavex ImpreSafe is a concentrate for the disinfection and cleaning of dental impressions made of alginate, polyether, polysulfide, A-silicone and C-silicone. Cavex ImpreSafe does not affect the dimensional stability of dental impressions. Free of aldehydes.

**INTENDED USERS:** The product is intended for use by dental or healthcare professionals with specific knowledge about the disinfection of medical devices. The product is not to be used directly on patients.

**EFFECTIVE SPECTRUM:** Bactericidal (EN 13272-2012+A2:2015\*, EN 14561:2006\*) 3% in 3 min.; yeasticidal (EN 13624:2013\*, EN 14562:2006\*) 3% in 3 min; enveloped viruses (EN 14476:2013+A1:2015\*) 3% in 3 min; limited virucidal (EN 14476:2013+A1:2015\*) 3% in 3 min; ^ Tested under organic load / # Tested under low and high organic load

**INSTRUCTIONS FOR USE:** For your protection, wear disposable gloves and safety goggles. Prepare the working solution according to the dosing table. Immerse dental impression and impression tray completely in the disinfectant solution. Adhere to but do not exceed indicated contact time. After disinfection, rinse impressions thoroughly under running water at least drinking quality for 10 seconds. Replace working solution daily or when visibly contaminated. A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use.

**LIMITATIONS:** FOR PROFESSIONAL USE ONLY. Not for use on invasive medical devices. Do not mix with other products. Make sure that all visible soiling has been removed before disinfection.

**HANDLING:** DANGER H302 Harmful if swallowed. H314 Causes severe skin burns and eye damage. H410 Very toxic to aquatic life with long lasting effects. P272 Wear protective/face protection/eye protection/face protection/face protection P301 + P312 IF SWALLOWED: Call a POISON CENTER or doctor/pharmacist if you feel unwell; P301 + P330 + P331 IF SWALLOWED: rinse mouth. Do NOT induce vomiting. P303 + P361 + P363 IF ON SKIN (or hair): Wash with water/shower. P303 + P351 + P338 IF ON EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

**STORAGE:** Store in cool, dry and well-ventilated facility. Keep out from direct sunlight, heat, and open flame. Storage temperature: < 30 °C

**COMPOSITION:** 100 g contains 30 g benzalkonium chloride, auxiliaries and water.

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